

## Vibrators

### **KV-36/ KV-26 plus/ KV-26/ KV-16**

(110-120 V / 50/60 Hz)



KV-36, KV-26 plus, KV-16  
stainless steel version  
plastic-powder coated  
(natural white close to RAL 9016)

KV-36, KV-26 plus, KV-16  
stainless steel version  
matt



**USER MANUAL**

**Dear customer,**

Thank you for choosing a product from the Wassermann range. Wassermann Dental-Maschinen incorporates the highest standards of quality and the latest technology. In order to enjoy maximum performance and years of trouble-free operation, please read this user manual carefully before you connect this device and start work, and operate the device according to the recommended guidelines. The operation safety and the functionality of this device can only be guaranteed if you follow both the general safety guidelines and the applying laws to prevent accidents as well as the precautions given in this user manual. We are not liable for any damages which occur due to inappropriate usage or faulty operation of this device.



Make sure that anyone using this device has read and understood this user manual.

Keep this user manual in a safe place where it can be referred to as required at any time.

 The unit complies with the relevant EU guidelines.



The unit is subject to the EU guidelines 2012/19/EU (WEEE Directive).

**Company address:**

Wassermann Dental-Maschinen GmbH

Rudorffweg 15-17

21031 Hamburg, Germany

Phone : +49 (0)40 730 926 -20/ -24 Fax.: +49 (0)40 730 37 24

export@wassermann-dental.com

www.wassermann.hamburg

www.facebook.com/WassermannDentalmaschinen

## Contents

|   |    |
|---|----|
| 1 Features.....   | 4  |
| 2 Safety Symbols used in this Manual.....                     | 4  |
| 3 Safety Guidelines.....                                      | 5  |
| 4 Responsibility for Operation or Damage.....                 | 5  |
| 5 Application.....  | 6  |
| 6 Before Starting.....  | 6  |
| 6.1 Transport.....  | 6  |
| 6.2 Installation.....   | 6  |
| 6.3 Storage.....  | 7  |
| 7 Installation/ Start-up.....                                 | 7  |
| 8 Operation.....  | 7  |
| 8.1 General Operating Instructions.....                       | 7  |
| 8.2 KV-36.....  | 8  |
| 8.3 KV-26 plus.....   | 8  |
| 8.4 KV-26.....  | 8  |
| 8.5 KV-16.....  | 8  |
| 9 Troubleshooting.....  | 9  |
| 10 Care and Maintenance.....                                  | 9  |
| 10.1 Cleaning.....  | 9  |
| 10.1.1 Cleaning the Rubber Plates.....                        | 10 |
| 10.2 Maintenance.....   | 10 |
| 10.3 Repairs.....   | 10 |
| 10.4 Spare Parts.....   | 10 |
| 10.5 Service Hotline 0049 (0)40 730 926 -20/ -24.....         | 10 |
| 10.6 Warranty.....  | 10 |
| 10.7 Scope of delivery/ Accessories.....                      | 11 |
| 11 Technical Data.....  | 12 |
| 12 Disposing of the Unit.....                                 | 14 |
| 12.1 Information on Disposal for Countries within the EU..... | 14 |
| 13 EU Declaration of Conformity.....                          | 15 |

## 1 Features

- **115 V version**
- Extremely powerful vibrator for even and void-free shaking
- Powerful vibration intensity thanks to the highest manufacturing accuracy
- Switchable from 3000 to 6000 oscillations
- **KV-26 plus** and **KV-36** with infinitely adjustable vibration intensity
- Even vibration intensity across the entire table, thanks to centrally mounted magnets suitable for continuous loads
- Removable and washable rubber supports
- All machines have a stainless steel yoke on the left-hand side which can be used when removing casts
- Extremely robust housing in stainless steel, plastic-powder coated or matt
- Heavy weight for maximum stability

## 2 Safety Symbols used in this Manual



### **Warning!**

This is a warning of risk situations and dangers.

Failure to observe this warning could be life-threatening. These warnings has to be observed.



### **Information!**

This symbol draws your attention to specific features that has to be observed.

### 3 Safety Guidelines

Configuring and operating this equipment requires **precise knowledge** and **observance** of the instructions in this **user manual**. The equipment is designed only for its intended application.



**WARNING:**

Servicing and repairs should be carried out only by authorised specialists.



The unit is designed to operate on 110-120 V / 50/60 Hz.

Make sure that the equipment is connected to the correct power source.

Shocks from handle.

The handle is not intended for carrying the unit. Its mounting points are not designed to take the weight of the unit, which could be dropped as a result.

### 4 Responsibility for Operation or Damage

The responsibility for operating the device lies exclusively with the owner or user if said device is incorrectly serviced, maintained or altered by persons not employed by an authorised dealer or if the device is used in a manner contrary to its specified purpose.

The unit has to be maintained and operated in accordance with this user manual.

Wassermann Dental-Maschinen GmbH is not responsible for damage arising from the **nonobservance** of these instructions.

Warranty and responsibility provisions contained in the sales and supply conditions of Wassermann Dental-Maschinen GmbH are not extended by these instructions.

## 5 Application

**KV units** are used exclusively in dental laboratories to provide a blister-free, homogenous compound which is the prerequisite for a smooth finish on casts. In addition, all vibrators are fitted on the left-hand side with a stainless steel handle which can be used when removing casts.

Only use the device for this type of application.

## 6 Before Starting

### 6.1 Transport

Before transporting the unit, ensure that it has been unplugged from the power socket. Make sure that it is packed correctly in order to avoid accidental damage.



Be sure to check for any transport damage when unpacking the goods. Note down any damage if found.



The handle is not intended for carrying the unit. Its mounting points are not designed to take the weight of the unit, which could be dropped as a result.

### 6.2 Installation

Open the box, remove the packing materials, and carefully lift out the device and accessories. Check the included accessories.

The device has to stand horizontally on a steady and even surface.



Install the device in a place where it will not block the working area and the functionality (take the dimensions into account).



Do not install the unit outdoors or in places without proper ventilation.

Before start-up, be sure the device reaches room temperature.

### 6.3 Storage

If the unit is to be stored for an extended period, protect it from moisture and dust.

## 7 Installation/ Start-up



Insert the power plug into the socket, making sure that the mains and the unit operate on the same voltage.

## 8 Operation

### 8.1 General Operating Instructions

All instructions for using the unit, whether in verbal or written form, are based on our own experience and experimentation and can only be regarded as guidelines.

Place the container on the vibrating table or attach it to the handle (the intensity is varied by pressing the container on the vibrating surface).

Note: When mixing materials or removing casts using vibrators, there will be almost zero air disturbance.

If the unit is not going to be used for an extended period, remove the plug from the socket.

## 8.2 KV-36

Switch the unit on:

Turn left-hand knob 1-0-2 to select the vibrations per minute.

(1 = 3000 vibrations/min. - 2 = 6000 vibrations/min.)

Select the vibration intensity using the right-hand knob (with the arrow).

(Turn left for lower intensity or right for increased intensity.)

## 8.3 KV-26 plus

Switch the unit on:

Turn left-hand knob 1-0-2 to select the vibrations per minute.

(1 = 3000 vibrations/min. - 2 = 6000 vibrations/min.)

Select the vibration intensity using the right-hand knob (with the arrow).

(Turn left for lower intensity or right for increased intensity.)

## 8.4 KV-26

Switch the unit on:

Turn knob 1-0-2 to select the vibrations per minute.

(1 = 3000 vibrations/min. - 2 = 6000 vibrations/min.)

## 8.5 KV-16

Switch the unit on:


Turn knob 1-0-2 to select the vibrations per minute.

(1 = 3000 vibrations/min. - 2 = 6000 vibrations/min.)



## 9 Troubleshooting

| Fault  | Cause   | Solution  |
|--|---|---|
| KV unit does not vibrate   | Power plug not connected                        | Insert and/or check plug  |
|  | Faulty knob or rocker switch                    | Replace or repair switch (contact our service department)                           |
| Vibrator does not vibrate or vibrates at the same level, i.e., the intensity is not variable<br><b>(KV-36 and KV-26 plus only)</b> | Faulty KV vibration regulator and potentiometer | Replace or repair potentiometer or entire assembly (contact our service department) |

 If the above recommendations do not solve the problem, contact your dental depot or our service department.

## 10 Care and Maintenance

### 10.1 Cleaning



Disconnect the power plug before starting any maintenance work.

The identification plate has always to be kept in easily legible condition and has not to be removed.



Remove external dirt from time to time with some form of cold cleaner.

Use only cold cleaners to avoid damaging the paintwork or removing the lettering. The equipment should be cleaned at regular intervals to ensure trouble-free operation.

It requires only normal cleaning (sponge, damp cloth, mild detergent) and no further chemical additives.

### 10.1.1 Cleaning the Rubber Plates

To make cleaning the rubber plates easier, they can simply be removed from the vibrating table and reinstalled after cleaning.

## 10.2 Maintenance

The unit does not require any servicing. Just make sure that the device is kept clean.

## 10.3 Repairs



Service or repairs to the unit has only to be carried out by qualified technicians. Only original spare parts are to be used. Responsibility for the product is voided if unauthorised persons alter it or if inappropriate components are installed.

## 10.4 Spare Parts

If necessary please contact our service hotline phone.

## 10.5 Service Hotline 0049 (0)40 730 926 -20/ -24

## 10.6 Warranty



The warranty period for our equipment is 12 months. If faults occur within the warranty period, contact your dental depot or get in touch directly with our service department.

Your equipment should only be operated in perfect condition. If faults occur which could harm operators or third parties, the unit should not be used until it has been fixed.

This warranty does not cover damage caused by improper use, external mechanical causes, transport damage or interference with the unit by unauthorized persons.

## 10.7 Scope of delivery/ Accessories

| Included parts:  | Item no. |
|--|----------|
| <b>KV-36 115 V version</b> , cpl. with rubber plate<br>stainless steel version<br>plastic-powder coated (natural white close to RAL 9016)      | 169991 A |
| <b>KV-26 plus 115 V version</b> , cpl. with rubber plate<br>stainless steel version<br>plastic-powder coated (natural white close to RAL 9016) | 169995 A |
| <b>KV-26 115 V version</b> , cpl. with rubber plate<br>stainless steel version<br>plastic-powder coated (natural white close to RAL 9016)      | 169993 A |
| <b>KV-16 115 V version</b> , cpl. with rubber plate<br>stainless steel version<br>plastic-powder coated (natural white close to RAL 9016)      | 169994 A |

| Accessories:                                | Item no. |
|---|----------|
| Spare rubber plate <b>KV-36</b>             | 169010 C |
| Spare rubber plate <b>KV-26 plus/ KV-26</b> | 169101 C |
| Spare rubber plate <b>KV-16</b>             | 169201 C |

**11 Technical Data**
**Power Vibrators**

|                     |   |
|---------------------|---|
| <b>KV-36</b>        | <b>Item no.: 169991 (plastic-powder coated)</b> |
| Voltage*            | 110-120 V / 50/60 Hz                            |
| Power consumption   | 1.2 A   |
| Output              | 136 W   |
| W x H x D           | 308 x 173 x 208 mm                              |
| Weight              | 10.9 kg   |
| Sound level         | 58 dB(A)  |
| Vibrating intensity | 3000 / 6000 vibr./min. (infinitely variable)    |
| Table dimensions    | 293 x 200 mm                                    |
| Safety class        | IP 54   |

|                     |   |
|---------------------|---|
| <b>KV-26 plus</b>   | <b>Item no.: 169995 (plastic-powder coated)</b>                   |
| <b>KV-26</b>        | <b>Item no.: 169993 (plastic-powder coated)</b>                   |
| Voltage*            | 110-120 V / 50/60 Hz  |
| Power consumption   | 0.6 A   |
| Output              | 70 W  |
| W x H x D           | 228 x 150 x 143 mm  |
| Weight              | 5.2 kg  |
| Sound level         | 52 dB(A)  |
| Vibrating intensity | 3000 / 6000 vibr./min. ( <b>KV 26-plus</b> : infinitely variable) |
| Table dimensions    | 210 x 130 mm  |
| Safety class        | IP 54   |

|                     |   |
|---------------------|---|
| <b>KV-16</b>        | <b>Item no.: 169994 (plastic-powder coated)</b> |
| Voltage*            | 110-120 V / 50/60 Hz                            |
| Power consumption   | 0.6 A   |
| Output              | 70 W  |
| W x H x D           | 145 x 150 x 143 mm                              |
| Weight              | 3.9 kg  |
| Sound level         | 53 dB(A)  |
| Vibrating intensity | 3000 / 6000 vibr./min.                          |
| Table dimensions    | 130 x 130 mm                                    |
| Safety class        | IP 54   |

\*Other voltages on request.

The noise level of the unit amounts to  $\leq 70$  dB (A).



Technical changes reserved.

## 12 Disposing of the Unit

The unit has to be disposed by an authorized recycling operation. The selected company has to be informed of all possibly health-hazardous residues in the unit.

### 12.1 Information on Disposal for Countries within the EU



To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authority.

**13 EU Declaration of Conformity**

in accordance with 2014/35/EU (Low Voltage Directive) and 2014/30/EU (Electromagnetic Compatibility Directive) and 2006/42/EC (Machinery Directive) and 2011/65/EU (RoHS Directive)

Manufacturer: **WASSERMANN** Product description: Vibrators for dental applications  
Dental-Maschinen GmbH  
Rudorffweg 15-17  
21031 Hamburg  
Germany

Model: KV-36 REF. 169991  
KV-26 plus REF. 169995  
KV-26 REF. 169993  
KV-16 REF. 169994

Applicable standards: DIN EN 61010-1  
DIN EN 61000-6-3  
DIN EN 61000-6-1  
DIN 45635-1  
DIN EN 60335-1

Hiermit wird bestätigt, dass die oben bezeichnete Maschine den genannten EU-Richtlinien entspricht. Diese Erklärung wird ungültig, falls die Maschine ohne unsere Zustimmung verändert wird.

This is to confirm that the above mentioned machine complies with the described EU guidelines. This declaration becomes invalid if the machine is modified without our approval.

Cette machine est conforme aux normes en vigueur d'Union européenne. Cet avis est nul et non avenant si cette machine est modifiée sans notre accord.

Esta máquina, anteriormente mencionada, cumple con los límites requeridos por el reglamento UE. Ahora bien, esta declaración quedará invalidada en caso de realizar modificaciones al aparato sin nuestra aprobación.

Hiermee wordt bevestigd dat bovengenoemde machine voldoet aan de voorgeschreven EU normen. Deze verklaring verliest geldigheid als er zonder onze uitdrukkelijke toestemming wijzigen aan de machine worden aangebracht.

Place, date: Hamburg, 2020-02-28

Company stamp :



Signature: \_\_\_\_\_  
Sven Wassermann  
(Managing Director)



**WASSERMANN**  
DENTAL-MASCHINEN GMBH  
Rudorffweg 15-17 · 21031 Hamburg



Wassermann Dental-Maschinen GmbH  
Rudorffweg 15–17 · 21031 Hamburg · Germany  
Phone: +49 (0)40 730 926-20/ -24 · Fax: +49 (0)40 730 37 24  
[export@wassermann-dental.com](mailto:export@wassermann-dental.com) · [www.wassermann.hamburg](http://www.wassermann.hamburg)



[/WassermannDentalmaschinen](https://www.facebook.com/WassermannDentalmaschinen)