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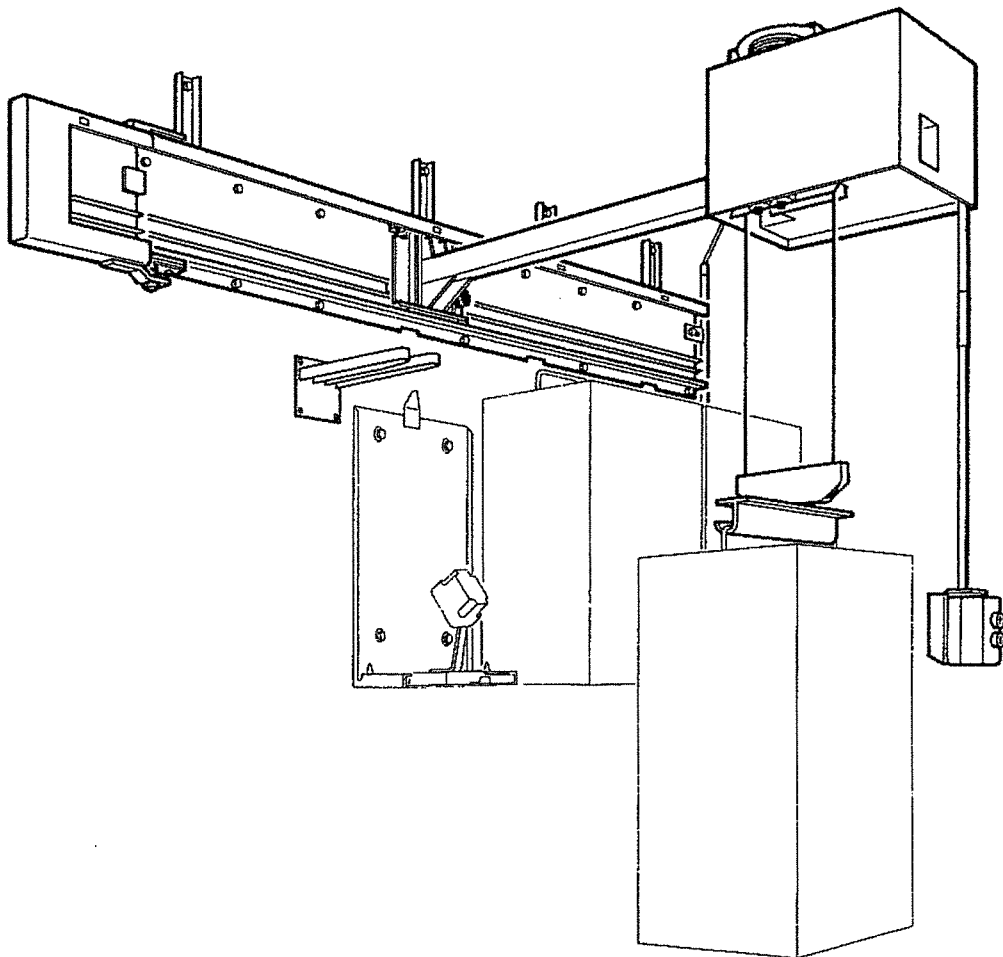
Dear customer,

Thank you for choosing a product from the Wassermann range. Wassermann dental equipment incorporates the highest standards of quality and the latest technology.

In order to enjoy maximum performance and years of trouble-free operation, please read this instruction manual carefully before you connect this device and start work, and operate the device according to the recommended guidelines. The operation safety and the functionality of this device can only be guaranteed if you follow both the general safety guidelines and the applying laws to prevent accidents as well as the precautions given in this instruction manual. We are not liable for any damages which occur due to inappropriate usage or faulty operation of this device.

Make sure that anyone using this device has read and understood this instruction manual.

Keep this instruction manual in a safe place where it can be referred to as required at any time.

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 The unit complies with the relevant EC guideline.

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directiv).

Disposing of the unit

The unit has to be disposed by an authorized recycling operation. The selected company has to be informed of all possibly health-hazardous residues in the unit.

Information on disposal for countries within the EC

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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4. Location

4.1 The required dimensions and weights must be taken into account.

⚠ Attention!
Check the load bearing capacity of the relevant wall!
Maximum load is 500 N (50 kp).

With Group I building materials

Wall thickness:

Concrete	200 mm
Bricks	115 mm
Solid limestone	240 mm

the set of screw supplied must be used.

With Group II building materials

Wall thickness:

Perforated bricks	115 mm
Porous bricks	115 mm
Perforated limestone	240 mm
Light concrete (pumice)	115 mm
Gas concrete	240 mm
Hollow blocks of light concrete	240 mm

injection dowel plugs Order No. 224 6666 and screw Order No. 251 0202 with washer Order No. 242 4022 must be used (not included in parts supplied).

Electric Dispenser Hoist 2

Min. ceiling height:	2530 mm
Min. wall length:	2196 mm
Weight:	75.0 kg

Electric Dispenser Hoist 3

Min. ceiling height:	2530 mm
Min. wall length:	2610 mm
Weight:	80.0 kg

Electric Dispenser Hoist 4

Min. ceiling height:	2530 mm
Min. wall length:	3016 mm
Weight:	85.0 kg

Electric Plaster Dispenser




30	
Weight empty:	approx. 26 kg
Weight full:	approx. 56 kg

Plaster Container 30

Weight empty:	approx. 16 kg
Weight full:	approx. 46 kg

Note

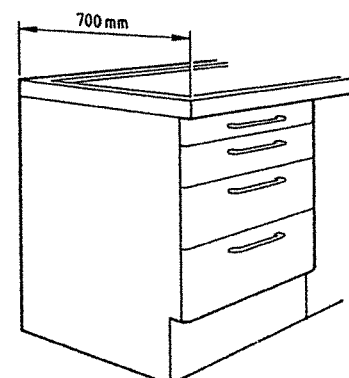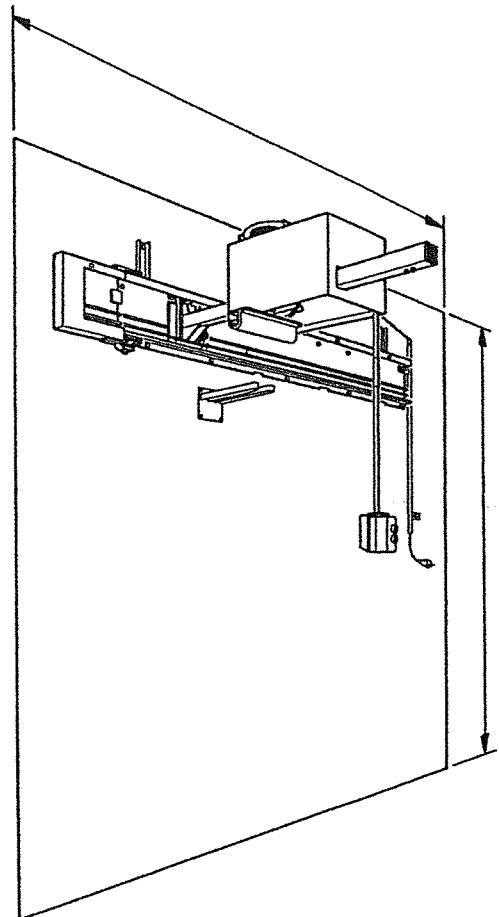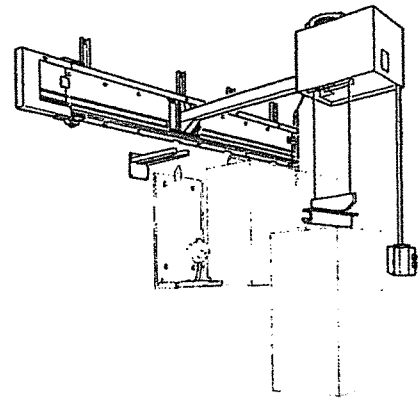Maximum depth of the table or cabinet under the EWL Electric Dispenser Hoist must not exceed 700 mm.

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5.6 Mounting the threading rail ④

5.6.1 Move the carriage ① in the direction of the arrow ◀ up to the limit stop.

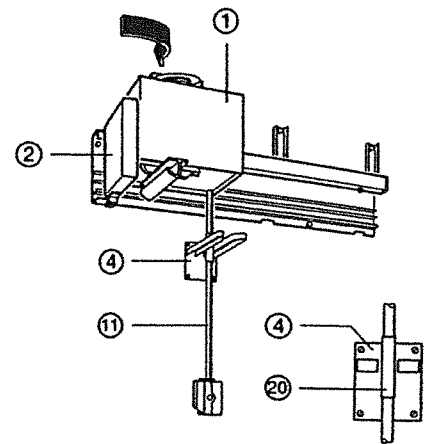
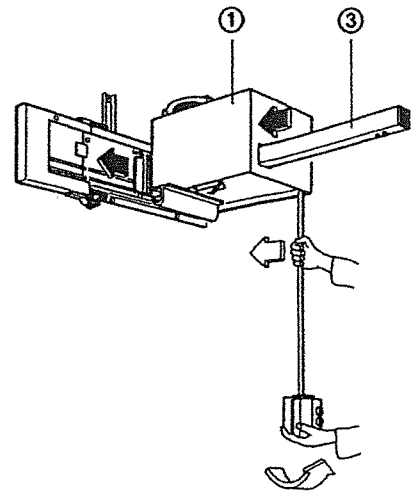
5.6.2 Lift the carriage ① and load arm ③ in the direction of the arrow ▶ using the support tube and slide them in the direction of the limit stop.

5.6.3 Swivel in the carriage ① with swivelling part of the guide rail ② in the direction of the arrow ▶ up to the limit stop (parking position).

5.6.4 Apply the threading rail ④ to the wall with the plastic protection ⑳ central to the support tube ⑪. Align horizontally/vertically and mark holes.

5.6.5 Centre drill the marked holes \varnothing 6 mm.

5.6.6 Insert dowel plugs and screw on the threading rail ④.

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8. Operation

Attention!
 The maximum loading capacity is 500 N (50 kp)! Do not exceed this limit on any account!
 Use only in accordance with regulations!
 Do not step under raised load or carriage ①!

8.1 Swivel out the carriage ① and load arm ③ in the direction of the arrow ◀ up to the limit stop on the guide rail. Slide the carriage ① in the direction of the arrow ◀ up to the limit stop on the guide rail ②.

8.1.1 Raise the support tube ⑩ in the direction of the arrow ▷ and move the load arm ③ right up to the desired position.

Note

The load arm ③ engages in the position central to the dispenser.

8.2 Lower the load hook ⑥ down to the cover stop ㉔ by operating the button ㉕.

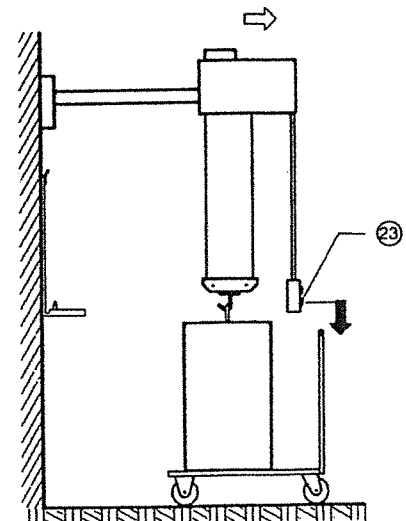
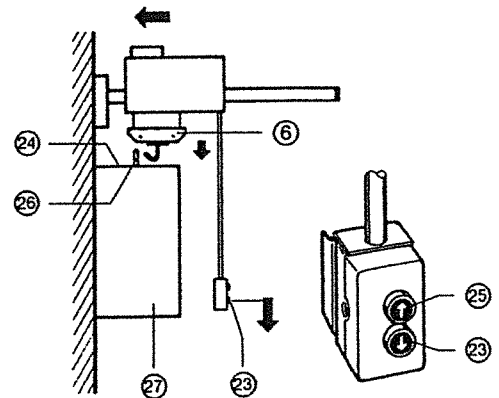
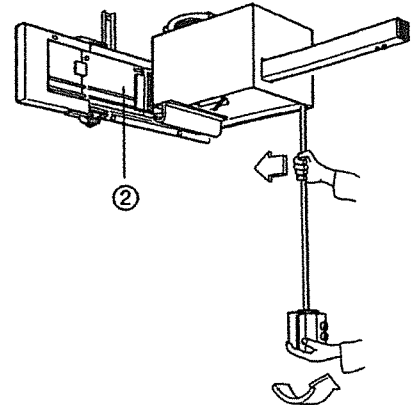
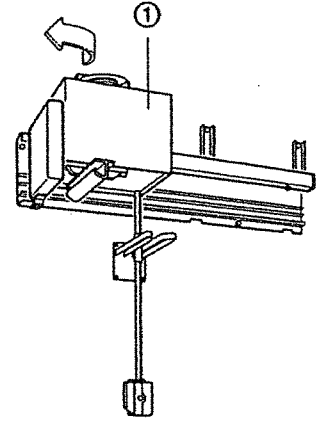
8.2.1 Raise the load hook ⑥ by pressing the button ㉕ thereby raising the plaster container ㉗ to the limit stop (automatic limit off-switching).

Note

The load hook ⑥ automatically hooks into the bracket ㉖.

8.2.2 Move the carriage ① forwards in the direction of the arrow ▷ and operate the button ㉕ to lower the container ㉗ down onto the EWL trolley or the floor.

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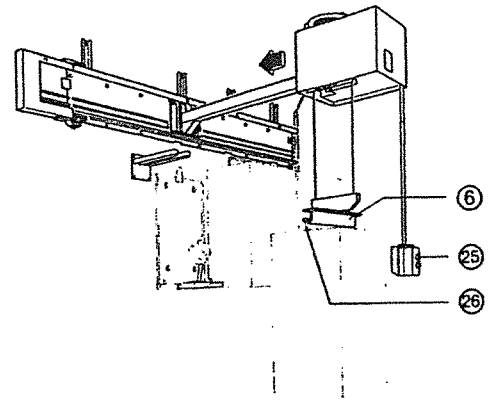
8.3 Hook the load hook ⑥ into the bracket ㉓ again and raise the full plaster container to the stop by operating the button ㉓ .

Note

Load hook ⑥ with suspended plaster container ㉗ is stopped at the upper limit stop by an automatic limit switch.

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8.3.1 Move the carriage ① in the direction of the arrow 4 to the rear limit stop.

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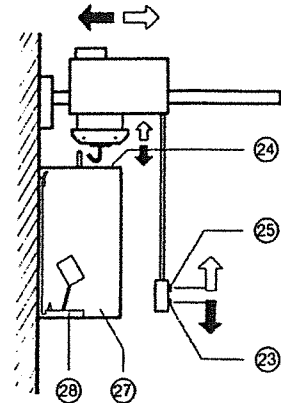
8.3.2 Operate the button ㉓ to lower the plaster container in the direction of the console ㉔ .

Attention!

During the lowering process, push the container ㉗ against the console ㉔ . The container ㉗ must engage fully in the supports of the console ㉔ .

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
8.3.3 Lower the load hook ⑥ to the cover ㉔ stop. Move the carriage ① slightly forwards and raise the load hook ⑥ to the stop by operating the button ㉓ .

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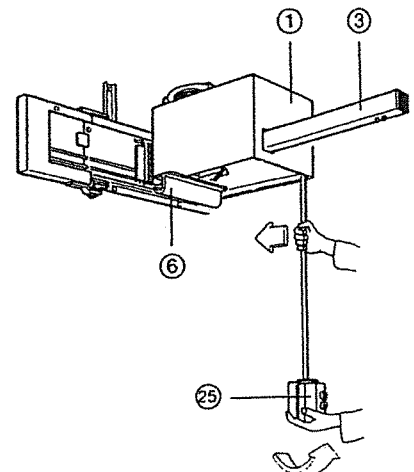8.3.4 Slide the carriage ① up to the limit stop on the guide rail ② . Raise the load arm ③ and move it to the left into the parking position.

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9. Maintenance

Attention!
Repairs and servicing work on the electrical part of this equipment must only be undertaken by experts or by persons trained in our factory who are aware of the safety regulations. Disconnect the mains plug from the power supply or switch off the disconnecting switch before opening any part of the equipment.

9.1 Carry out a monthly check on the steel rope ⑤ and the winding tracks. Check the power cord for wear or damage.

Attention!
In the event of wire breakages in the steel rope ⑤, this must be replaced for safety reasons.

10. Operating faults

10.1 Power cord ⑨ does not coil up automatically

10.1.1 Possible causes

- a) Power cord ⑨ jammed
- b) Cable pulley ⑳ defective

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
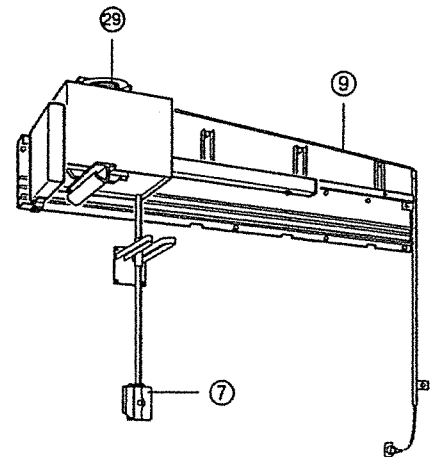
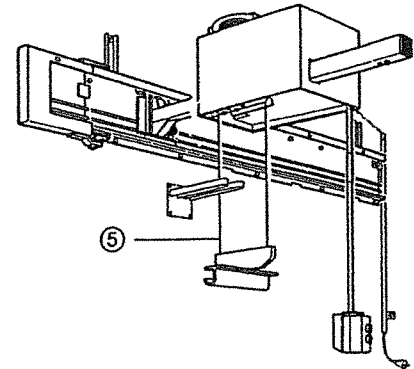
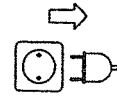
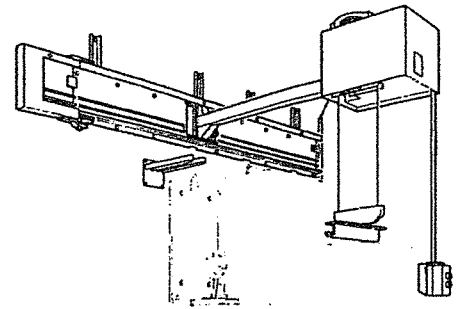
We reserve the right to make any alterations.

11. Caractéristiques techniques	11. Datos técnicos	Für Ihre Notizen. Pour vos remarques.	For your notes. Para sus notas.
Cotes d'encombrement	Dimensiones:		
Longueur:	Longitud:		
Ascenseur . 2 1540 mm	Elevador de silos 2 1540 mm		
Ascenseur . 3 1950 mm	Elevador de silos 3 1950 mm		
Ascenseur . 4 2360 mm	Elevador de silos 4 2360 mm		
Profondeur:	Profundidad:		
Ascenseur 2/3/4 1150 mm	Elevador de silos 2/3/4 1150 mm		
Poids:	Peso:		
Ascenseur 2 75 kg	Elevador de silos 2 75 kg		
Ascenseur 3 80 kg	Elevador de silos 3 80 kg		
Ascenseur . 4 85 kg	Elevador de silos 4 85 kg		
Puissance branchée absorbée: 200 W	Potencia conectada: 200 W		
Capacité de charge maximale: 500 N 50 kgf	Capacidad máxima de sustentación: 500 N 50 kp		
Tous droits de modifications techniques réservés.	Reservado el derecho de modificaciones técnicas.		

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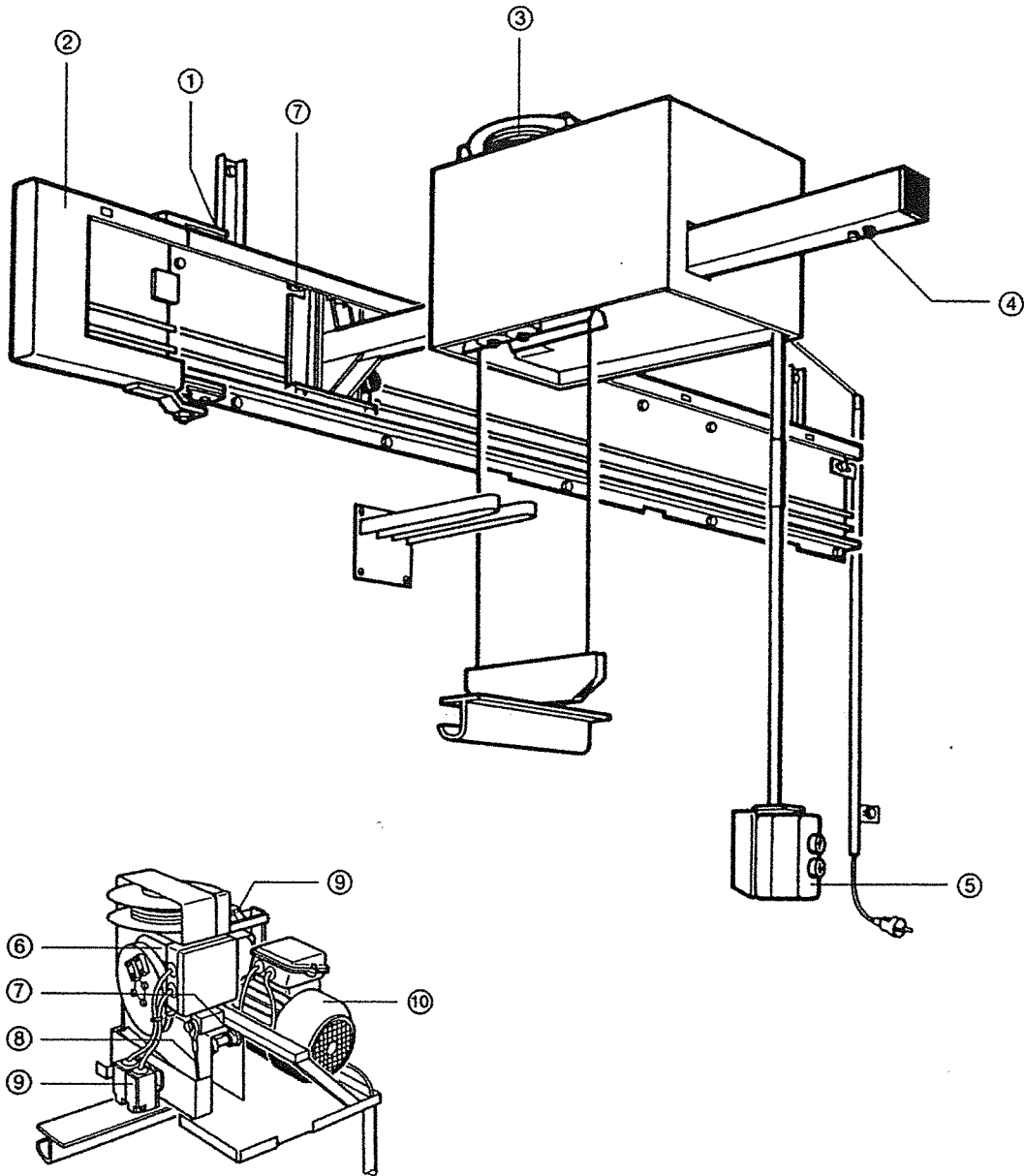Availability of spare parts is guaranteed over the long term.

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Wassermann Dental-Maschinen GmbH