Schleifscheibenwechsel

HSS-99

Diamantschleifscheibe

Replacement of the Grinding Disc

HSS-99

Diamond Grinding Disc

Changement de la meule

HSS-99

La meule diamantée

- (DE)** BETRIEBSANLEITUNG
- (GB)** USER MANUAL
- (FR)** MODE D'EMPLOI

(DE) Firmenanschrift:

Wassermann Dental-Maschinen GmbH

Rudorffweg 15-17

21031 Hamburg, Deutschland

Tel.: +49 (0)40/ 730 926 - 0 Fax.: +49 (0)40/ 730 37 24

sales@wassermann-dental.com

service@wassermann-dental.com

www.wassermann.hamburg

www.facebook.com/WassermannDentalmaschinen

Service-Hotline 040/ 730 92 6-50

(GB) Company address:

Wassermann Dental-Maschinen GmbH

Rudorffweg 15-17

21031 Hamburg, Germany

Phone : +49 (0)40 730 926 -20/ -24 Fax.: +49 (0)40 730 37 24

export@wassermann-dental.com

www.wassermann.hamburg

www.facebook.com/WassermannDentalmaschinen

Service Hotline 0049 (0)40 730 926 -20/ -24

(FR) Adresse :

Wassermann Dental-Maschinen GmbH

Rudorffweg 15-17

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Tél. : +49 (0)40 730 926 -20/ -24 Fax. : +49 (0)40 730 37 24

export@wassermann-dental.com

www.wassermann.hamburg

www.facebook.com/WassermannDentalmaschinen

Service d'aide : 0049 (0)40 730 92 6-24

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GB Included in delivery/ Accessories

Included in delivery	Item no.
Spare diamond grinding disc, incl. puller and grease	164063

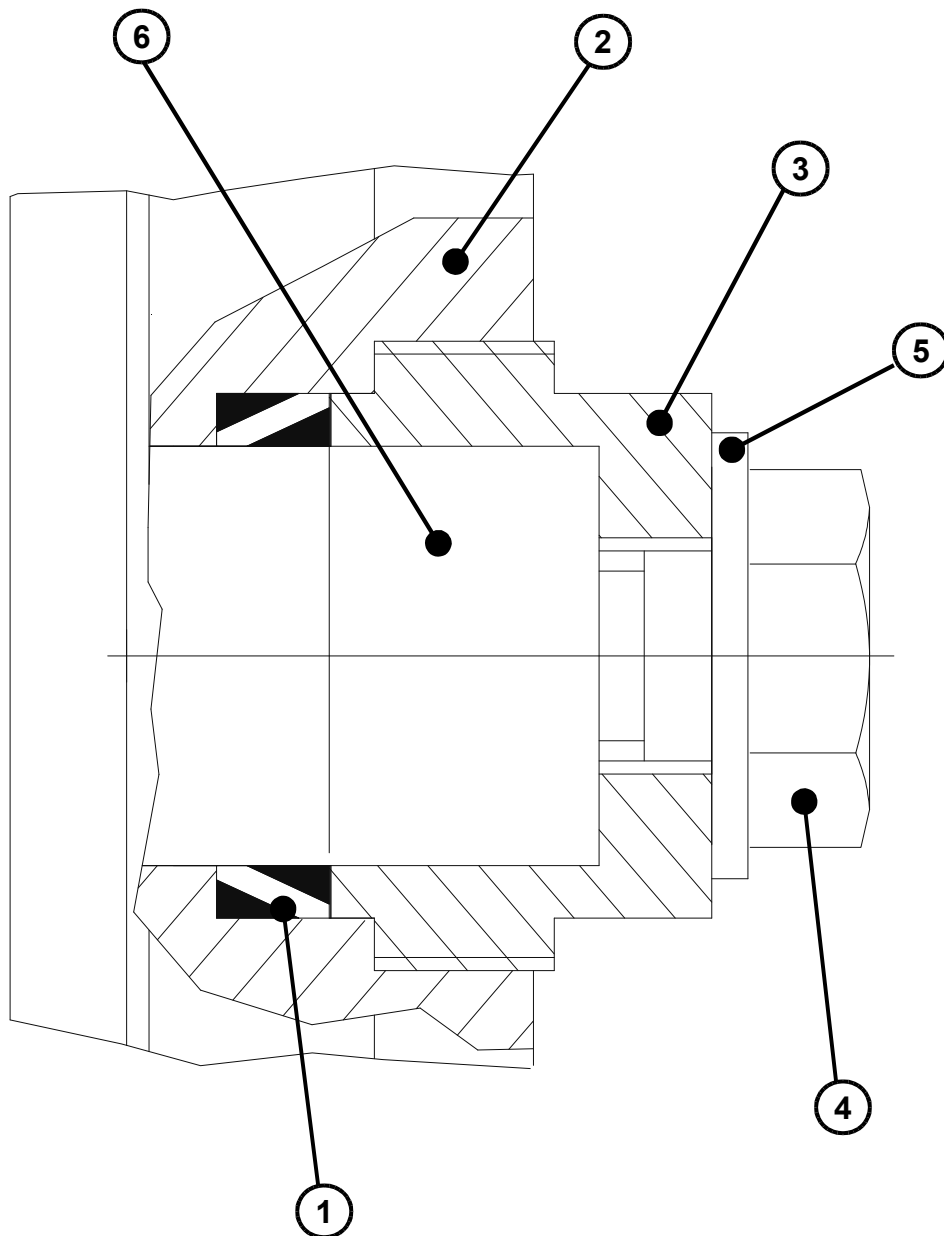
Accessories	Item no.
Sealing grease for trimmer, 25 ml	629002
Puller for grinding disc change	164026

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Replacement of the Diamond Grinding Disc HSS-99

WARNING:

Disconnect the power plug before starting any maintenance work.

1. Remove frontplate by unscrewing the star grips on the back.
2. Remove screw with disc from the shaft.
3. Unscrew the pressure bushing.
4. Mount the puller for removing the old disc.
5. Grease the motor shaft (***not*** the first 25 mm from the front, because the grip-spring tensioning element must be strong fitted).
6. Push the new diamond disc on the motor shaft up to the fixing.
7. Mount the grip-spring tensioning element ***without*** grease.
(Attention: fitting position!)
8. Thread the pressure bushing (grease the winding a little bit) into the diamond disc and fix (**clamp the grip-spring tensioning element!**).
9. Thread the screw (grease the winding) with disc into the end of the shaft and screw up.
10. Clean the surfaces of the front-plate and bearing box and grease with the delivered grease. Do not spread it.
11. Put back the front plate and fasten the star handles tightly. First the bottom star handles, then the top star handles. Now spread the grease and remove any grease.

Please also use the delivered grease for greasing the windings!

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