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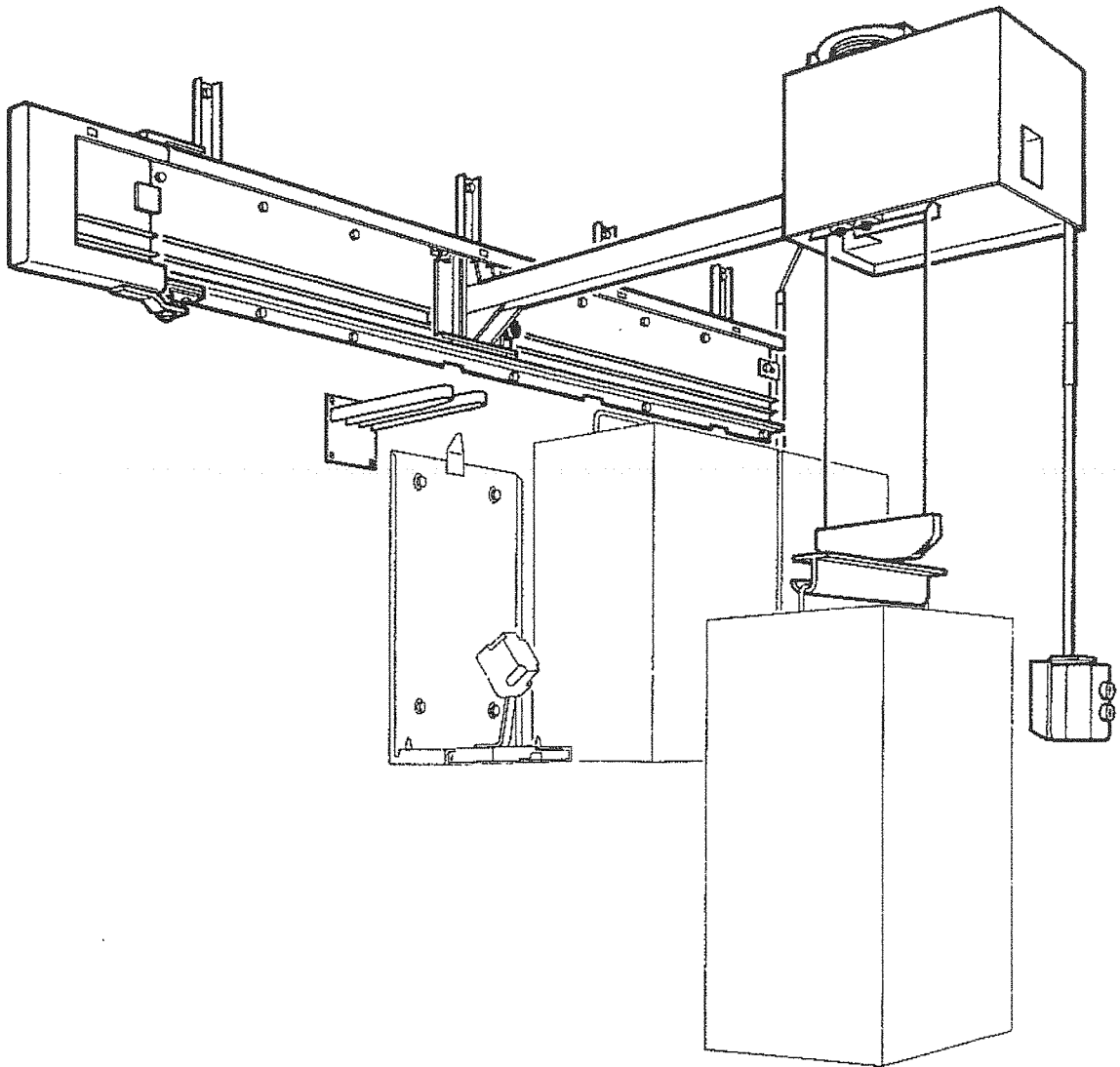
Dear customer,

Thank you for choosing a product from the Wassermann range. Wassermann dental equipment incorporates the highest standards of quality and the latest technology.

In order to enjoy maximum performance and years of trouble-free operation, please read this instruction manual carefully before you connect this device and start work, and operate the device according to the recommended guidelines. The operation safety and the functionality of this device can only be guaranteed if you follow both the general safety guidelines and the applying laws to prevent accidents as well as the precautions given in this instruction manual. We are not liable for any damages which occur due to inappropriate usage or faulty operation of this device.

Make sure that anyone using this device has read and understood this instruction manual.

Keep this instruction manual in a safe place where it can be referred to as required at any time.

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1. Index

	page
2. Scope of delivery—Accessories	2
3. Mains voltage—Mains frequency	2
4. Location	4
5. Mounting and connection	6-10
6. Preparations for commencing operation	12
7. Controls and functional elements	12
8. Operation	14-16
9. Maintenance	18
10. Operating faults	18
11. Technical data	20
12. Spare parts list	22-23

2. Scope of delivery—Accessories

2.1 Check the equipment schedule for discrepancies.

2.1.1 Electric Dispenser Hoist comprising:

- 1 Load arm
- 1 Carriage
- 1 Protective tube
- 1 Power plug
- 1 Threading rail
- 1 Set of screws
- 1 Mounting rail

2.1.2 Accessories required

- Electric Plaster Dispenser
- 1 Guide rail

2.1.3 Accessories supplied on request

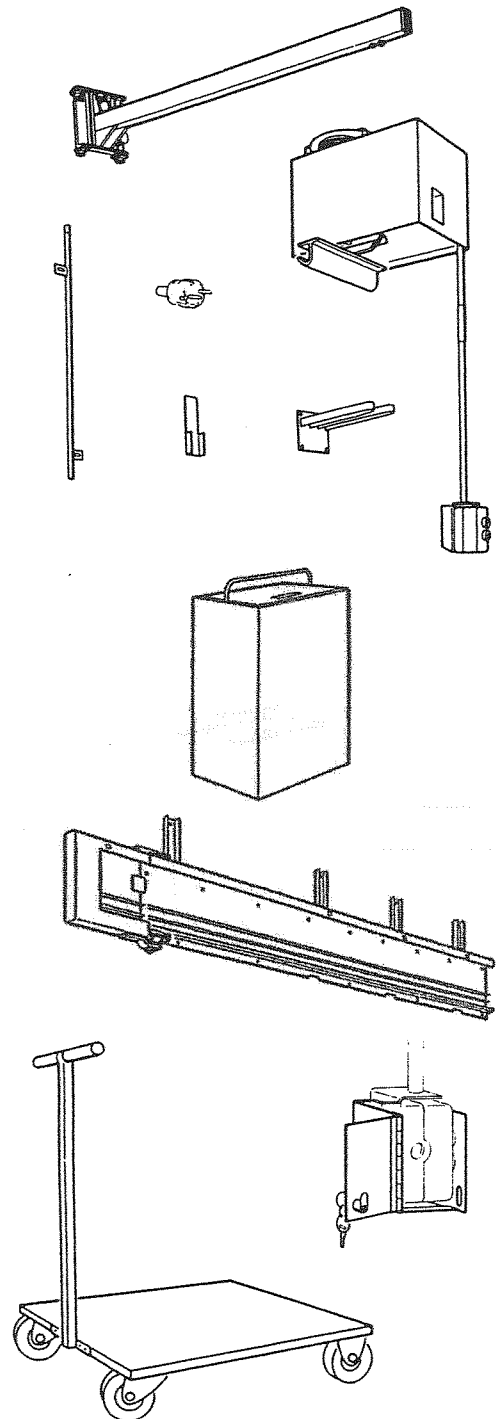
- Lock case
- Ord.No. 659 3361

- Trolley
- Ord.No. 659 4460

3. Mains voltage—Mains frequency

3.1 Electrical connection

Check that the local mains voltage and mains frequency coincide with the specifications on the rating plate.

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4. Location

4.1 The required dimensions and weights must be taken into account.

Attention!

Check the load bearing capacity of the relevant wall!
Maximum load is 500 N (50 kp).

With Group I building materials

Wall thickness:

Concrete	200 mm
Bricks	115 mm
Solid limestone	240 mm

the set of screw supplied must be used.

With Group II building materials

Wall thickness:

Perforated bricks	115 mm
Porous bricks	115 mm
Perforated limestone	240 mm
Light concrete (pumice)	115 mm
Gas concrete	240 mm
Hollow blocks of light concrete	240 mm

injection dowel plugs Order No. 224 6666 and screw Order No. 251 0202 with washer Order No. 242 4022 must be used (not included in parts supplied).

Electric Dispenser Hoist 2

Min. ceiling height:	2530 mm
Min. wall length:	2196 mm
Weight:	75.0 kg

Electric Dispenser Hoist 3

Min. ceiling height:	2530 mm
Min. wall length:	2610 mm
Weight:	80.0 kg

Electric Dispenser Hoist 4

Min. ceiling height:	2530 mm
Min. wall length:	3016 mm
Weight:	85.0 kg

Electric Plaster Dispenser




30	
Weight empty:	approx. 26 kg
Weight full:	approx. 56 kg

Plaster Container 30

Weight empty:	approx. 16 kg
Weight full:	approx. 46 kg

Note

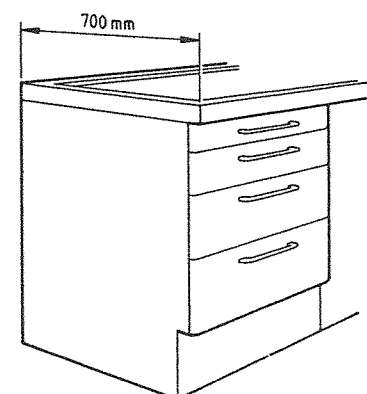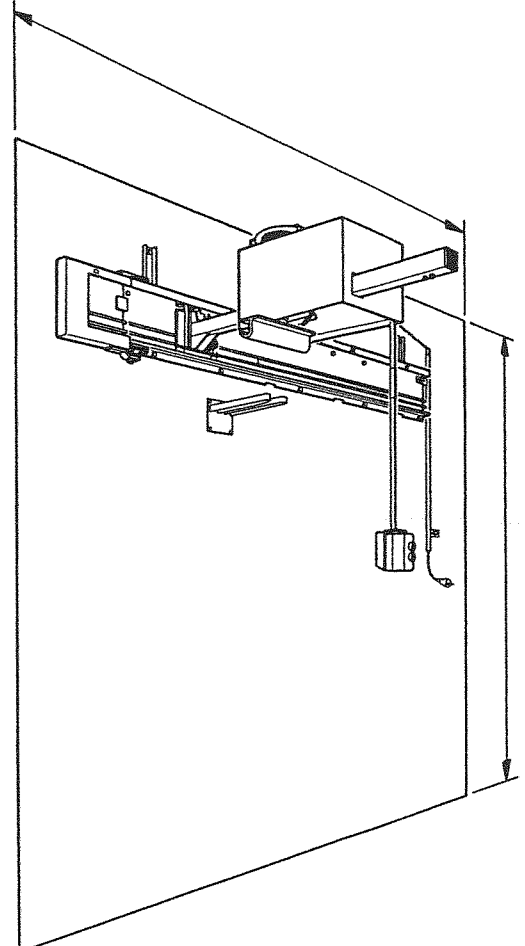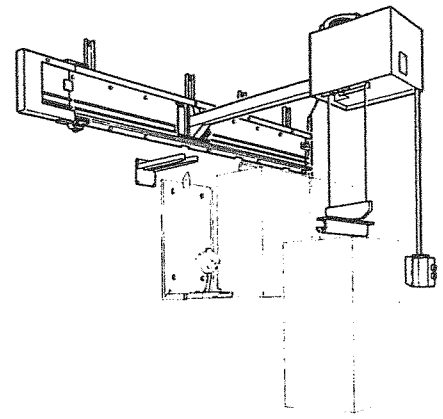Maximum depth of the table or cabinet under the EWL Electric Dispenser Hoist must not exceed 700 mm.

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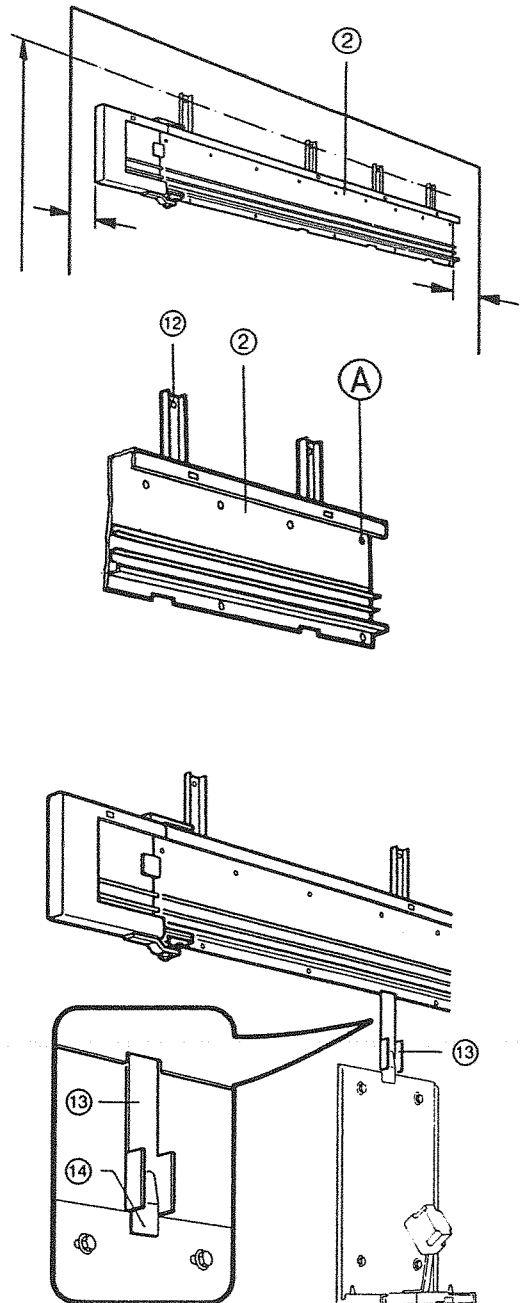
- 5. Mounting and connection**
- 5.1 Mounting the guide rail
- 5.1.1 Mark the top installation level (= 2500 mm) on the wall.
- 5.1.2 Hold the guide rail ② with its top holes ⑫ against this line and mark the holes.
- ⚠ Attention!**
Please observe the lateral minimum wall distances (right 150 mm, left 100 mm)!
- 5.1.3 Centre drill the marked holes $\varnothing 10 \times 90$ mm and insert dowel plugs.
- 5.1.4 Screw on the guide rail ② except for screw ① and align it horizontally.
- 5.2 Later mounting of the guide rail with EWL Plaster Dispenser 30 already fitted.
- 5.2.1 Fit the mounting strip ⑬ in the top guide bracket ⑭ of the dispenser console.
- 5.2.2 Apply the guide rail ② horizontally and insert the groove ⑮ in the mounting strip.
- 5.2.3 Align the guide rail ② horizontally and fasten it to the wall (see 5.1.2-5.1.4).
- 5.3 Mounting on an out-of-line or uneven wall.
- 5.3.1 Spacers must be used in accordance with the following tables to achieve the correct alignment.
- 5.3.2 Group I building materials


Zwischenlage	mm		Bemerkung	Spacer	mm		Remarks
Scheibe 11x3x34	0-3	Holzschraube 7 x 105 B.Nr. 247 2130	Im Liefer- umfang enthalten.	Washer 11x3x34	0-3	Wood screw 7 x 105 O.No.247 2130	In parts supplied
	6-12				6-12		
Best.Nr. 244 7005	15-24	Holzschraube 7 x 120 B.Nr. 247 2135	Nicht im Lie- ferumfang enthalten.	Order No. 244 7005	15-24	Wood screw 7 x 120 O.No. 247 2135	Not in parts supplied
	27-33				27-33		

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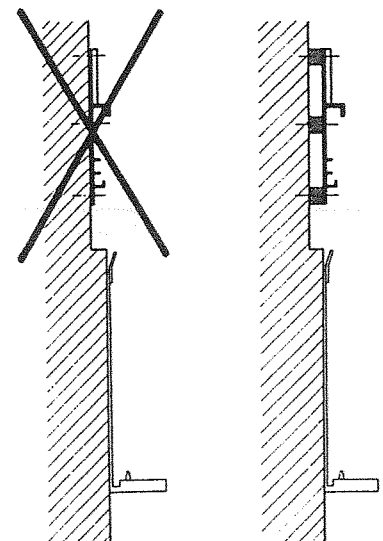5.4 Mounting the load arm

5.4.1 Insert the load arm ③ into the guide rail ② at the side.

5.4.2 Screw the protective tube ⑩ with the guide rail ② (hole A) to the wall.

Note

Please observe the minimum distance of 10 mm to the outer edge of the guide rail ②.

5.4.3 Align the protective tube ⑩ vertically, drill the lower securing point ③⑩ and screw to the wall.

Note

The protective tube ⑩ is a safety limit stop against accidental withdrawal of the load arm ③.

5.5 Mounting the carriage ①

5.5.1 Unscrew rubber buffer ⑰ and screws ⑱ on load arm ③.

5.5.2 Move the carriage ① with groove ⑱ in the direction of the arrow ④ along the load arm ③.

5.5.3 Screw on the rubber buffer ⑰ and screws ⑱ again.

Note

The rubber buffer ⑰ and screws ⑱ are limit and safety stops for the carriage.

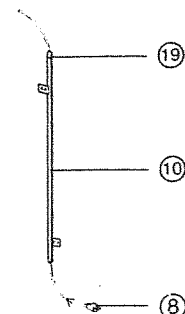
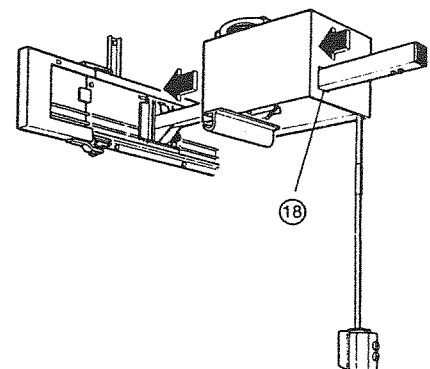
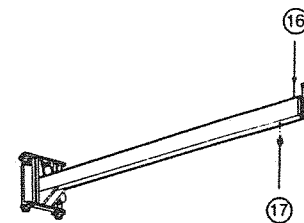
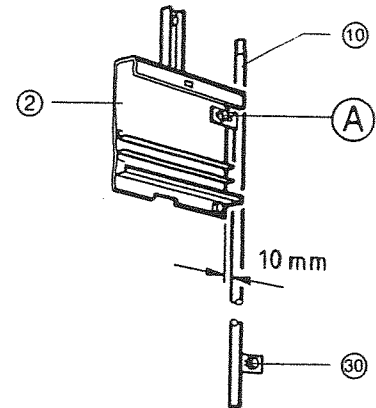
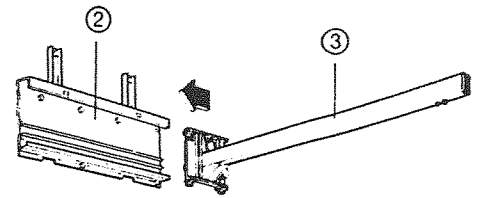
5.5.4 Unscrew screws ⑲ on protective tube ⑩ and pull through the power cord until the nearest socket outlet can be reached.

Attention!

Work on the electrical part of the equipment may only be carried out by specialists or staff trained in the factory, who are familiar with the safety regulations!

5.5.5 Connect the power plug ⑧ according to regulations.

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5.6 Mounting the threading rail ④

5.6.1 Move the carriage ① in the direction of the arrow ◀ up to the limit stop.

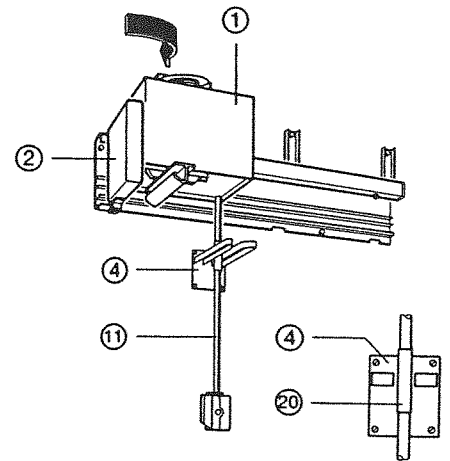
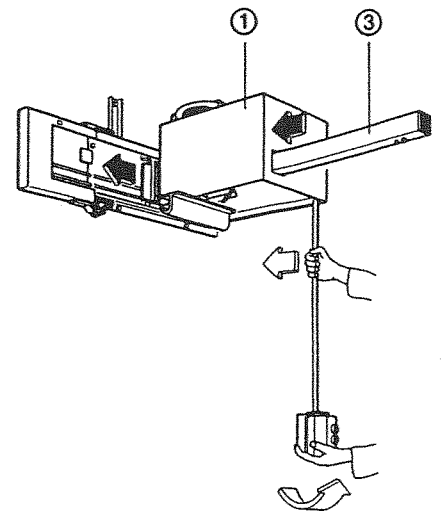
5.6.2 Lift the carriage ① and load arm ③ in the direction of the arrow ▶ using the support tube and slide them in the direction of the limit stop.

5.6.3 Swivel in the carriage ① with swivelling part of the guide rail ② in the direction of the arrow ▶ up to the limit stop (parking position).

5.6.4 Apply the threading rail ④ to the wall with the plastic protection ⑳ central to the support rail ⑪. Align horizontally/vertically and mark holes.

5.6.5 Centre drill the marked holes \varnothing 6 mm.

5.6.6 Insert dowel plugs and screw on the threading rail ④.

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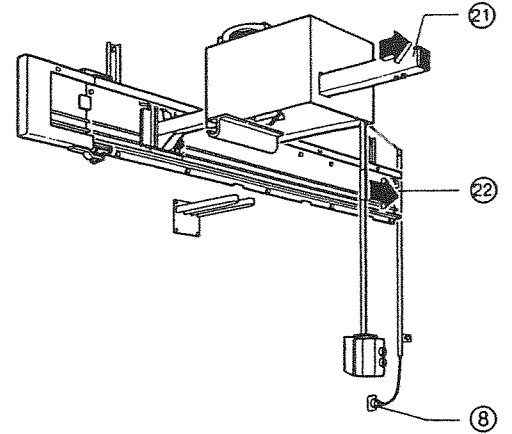
6. Preparations for commencing operation

6.1 Insert the power plug ⑧ in the correctly fused socket outlet provided.

6.2 Check the following functions:

The carriage ① must move smoothly without hindrance to the limit stop ②① of the load arm ③.

The carriage ①, presently against the limit stop on the load arm ③, must move without hindrance up to the limit stop ②② of the guide rail ②.

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
7. Controls and functional elements

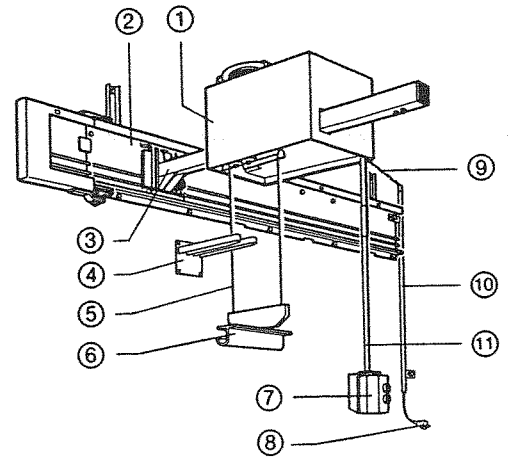- 1 Carriage
- 2 Guide rail
- 3 Load arm
- 4 Threading rail
- 5 Steel rope
- 6 Load hook
- 7 UP/DOWN button
- 8 Power plug
- 9 Power cord
- 10 Protective tube
- 11 Support tube

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8. Operation

Attention!

The maximum loading capacity is 500 N (50 kp)! Do not exceed this limit on any account!
Use only in accordance with regulations!
Do not step under raised load or carriage ①!

8.1 Swivel out the carriage ① and load arm ③ in the direction of the arrow ◁ up to the limit stop on the guide rail. Slide the carriage ① in the direction of the arrow ◁ up to the limit stop on the guide rail ②.

8.1.1 Raise the support tube ⑩ in the direction of the arrow ▷ and move the load arm ③ right up to the desired position.

Note

The load arm ③ engages in the position central to the dispenser.

8.2 Lower the load hook ⑥ down to the cover stop ㉔ by operating the button ㉓.

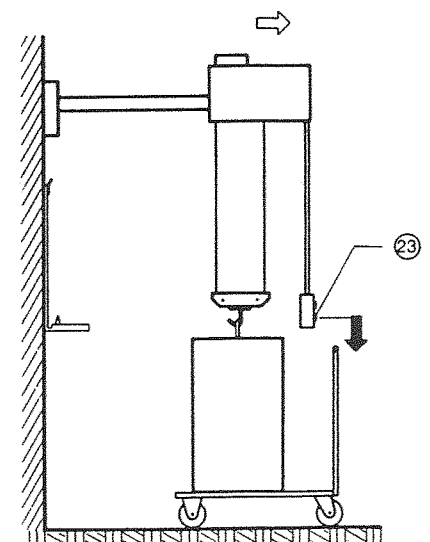
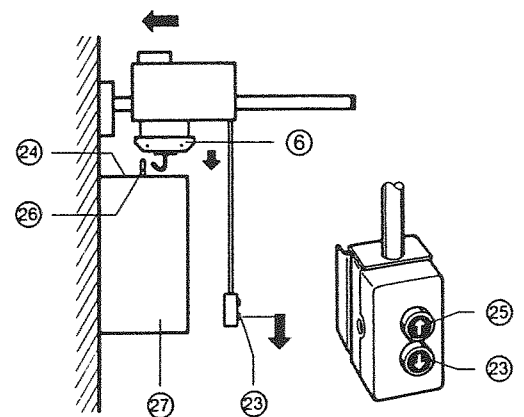
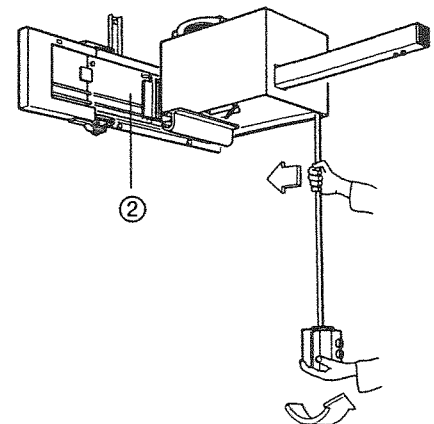
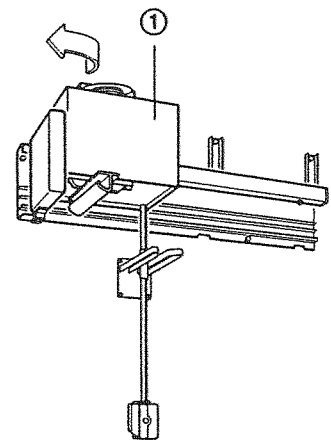
8.2.1 Raise the load hook ⑥ by pressing the button ㉕ thereby raising the plaster container ㉗ to the limit stop (automatic limit off-switching).

Note

The load hook ⑥ automatically hooks into the bracket ㉖.

8.2.2 Move the carriage ① forwards in the direction of the arrow ▷ and operate the button ㉕ to lower the container ㉗ down onto the EWL trolley or the floor.

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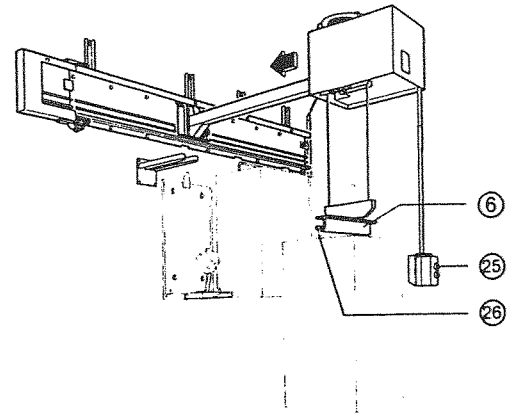
8.3 Hook the load hook ⑥ into the bracket ㉓ again and raise the full plaster container to the stop by operating the button ㉓ .

Note

Load hook ⑥ with suspended plaster container ㉓ is stopped at the upper limit stop by an automatic limit switch.

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8.3.1 Move the carriage ① in the direction of the arrow 4 to the rear limit stop.

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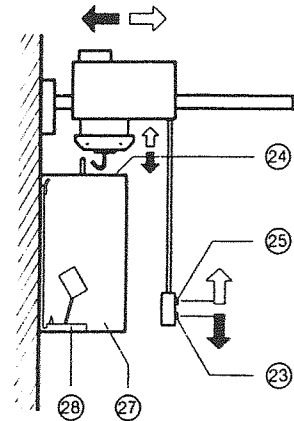
8.3.2 Operate the button ㉓ to lower the plaster container in the direction of the console ㉔ .

Attention!

During the lowering process, push the container ㉓ against the console ㉔ . The container ㉓ must engage fully in the supports of the console ㉔ .

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
8.3.3 Lower the load hook ⑥ to the cover ㉒ stop. Move the carriage ① slightly forwards and raise the load hook ⑥ to the stop by operating the button ㉓ .

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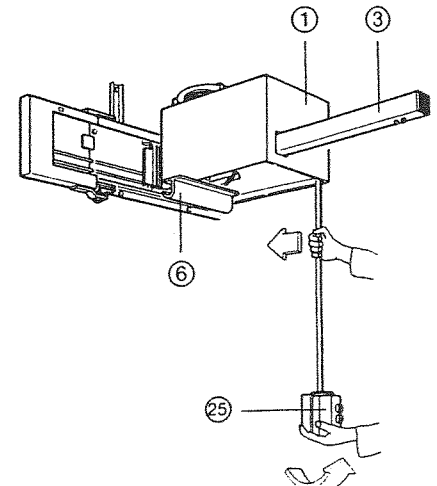8.3.4 Slide the carriage ① up to the limit stop on the guide rail ② . Raise the load arm ③ and move it to the left into the parking position.

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9. Maintenance

Attention!

Repairs and servicing work on the electrical part of this equipment must only be undertaken by experts or by persons trained in our factory who are aware of the safety regulations. Disconnect the mains plug from the power supply or switch off the disconnecting switch before opening any part of the equipment.

- 9.1 Carry out a monthly check on the steel rope ⑤ and the winding tracks. Check the power cord for wear or damage.

Attention!

In the event of wire breakages in the steel rope ⑤, this must be replaced for safety reasons.

10. Operating faults

- 10.1 Power cord ⑨ does not coil up automatically

10.1.1 Possible causes

- a) Power cord ⑨ jammed
b) Cable pulley ⑳ defective

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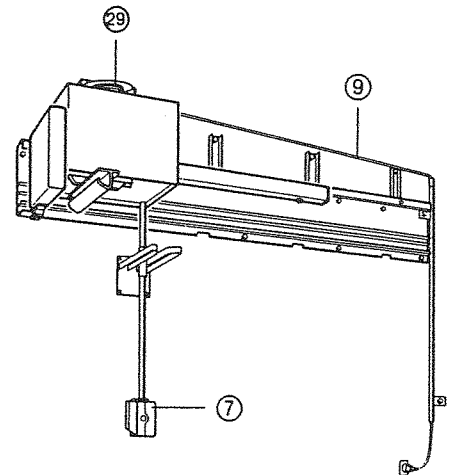
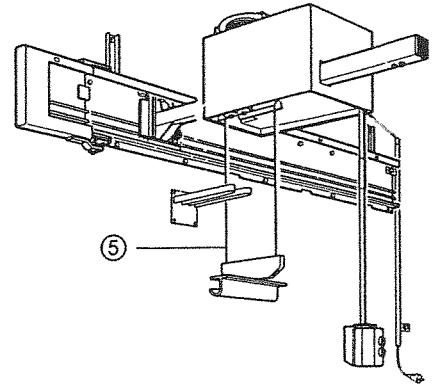
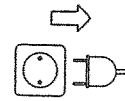
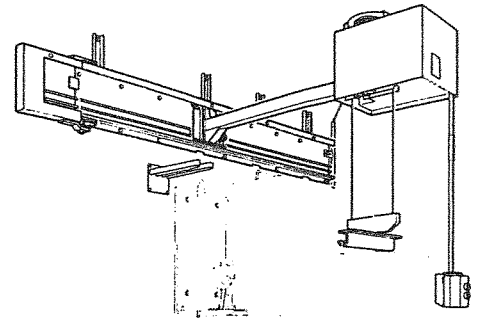
We reserve the right to make any alterations.

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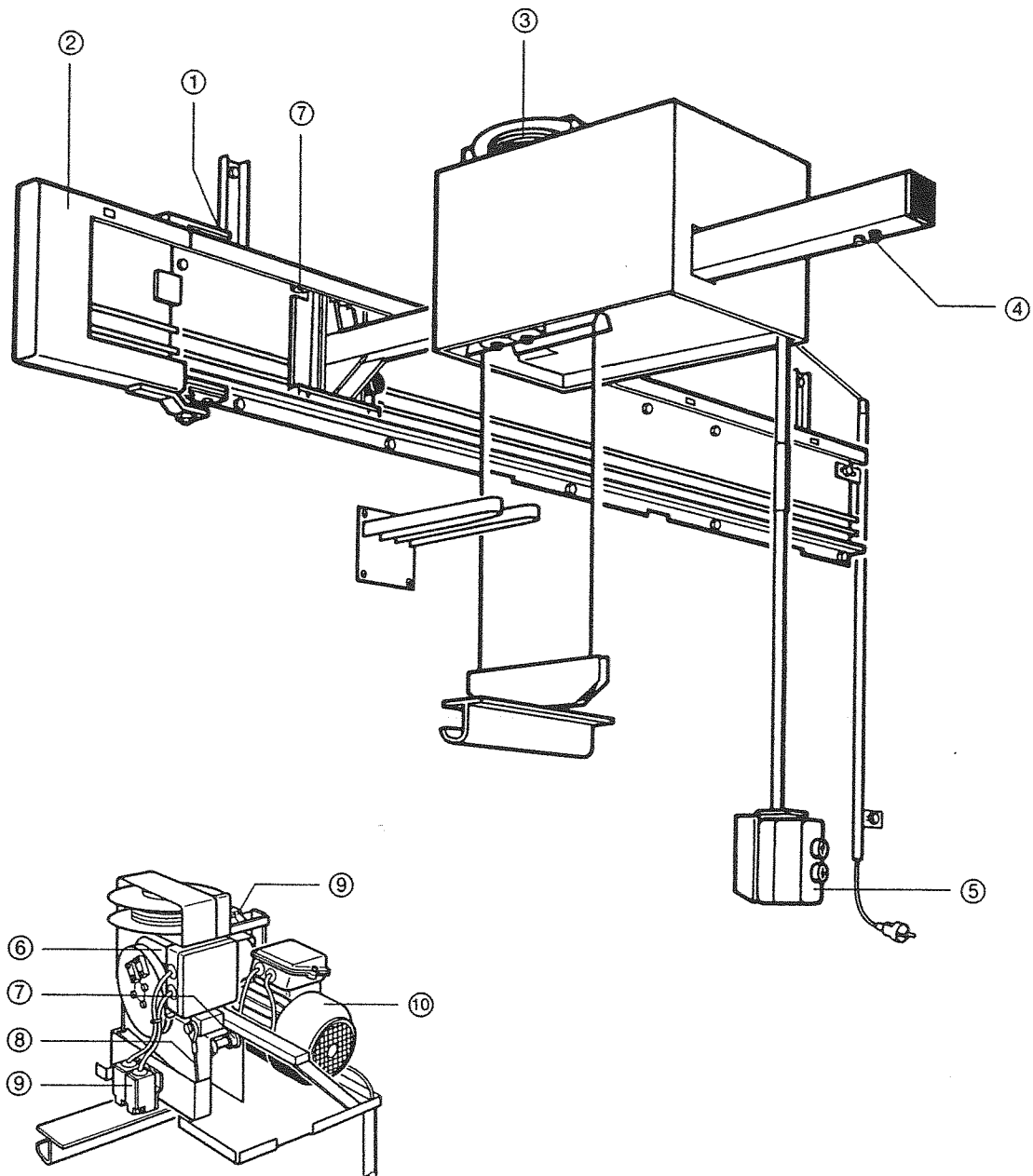
Availability of spare parts is guaranteed over the long term.

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This is to confirm that the above mentioned machine complies with the described EC rules. This declaration becomes invalid if the machine is modified without our approval.

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Dental-Maschinen GmbH Electric Dispenser Hoist
Wassermann Dental-Maschinen GmbH