

## Wasi-Steam 2 (REF 174992) Instructions for Flushing/ Decalcifying (to prevent Scale)

Please decalcify before operating the machine!

- Unit has to be in cold state (next day)
- Put the machine with the drain on the edge of the bench
- Have a bucket ready
- Open the discharge valve with the square socket wrench
- Connect to water supply
- Turn the machine on
- Press the black flushing button until only clear water comes out
  
- Close the discharge valve
- Fill the unit with water by pressing the flushing button until the manometer shows approx. 3 bar
- Put the bucket in front of the drain and open the discharge valve
- Move the unit towards the drain, so all water and dirt can come out
- Turn off the machine and open the discharge valve
  
- Do it again, this time flush with open discharge valve
  
- Close the discharge valve and start operating the machine according to the instructions in the manual

If you have any questions please email or call us

[service@wassermann-dental.com](mailto:service@wassermann-dental.com)

+49 40 730 926 -50



By decalcifying the machine it is ensured that all particles that might be loosen during transport will get out of the system and will not damage the seal (plunger) in the magnetic valve.



Before operating the unit you should read the manual instructions.

You can download the manual instructions here

<https://www.wassermann-dental.com/en.html>

