

Startup LSG-02 D (145998)/ LSG-02 DE (145997)

Assembly

To mount the unit, first pull the drawer slowly out of the holding frame until it stops.

Then carefully push the black plastic hook visible on the side of the rails in direction of arrow.

Now the frame can be removed from the suction drawer.

Attach the frame to the table plate.

The front screw hole should be 110 mm from the front edge of the plate.

Fasten the frame with 4 x M5 metal screws or 5 mm strong wood screws.

Then carefully insert the drawer into the guide rails of the mounted frame until the plastic hooks click into place, thus limiting the drawer extension.

Operation

Insert the power plug into the socket, making sure that the mains and the unit operate on the same voltage.

1. Press the (green) rocker switch.
2. A microswitch starts the extractor and any attached finishing motor as soon as the draw is pulled out.

How to set the automatic switch-on facility of LSG-02 DE

1. Pull the drawer completely out of the mounting frame. On the upper site of the unit there is a plastic blank plug, under which the potentiometer for setting is situated.
2. Switch on the Suction-drawer, switch on the Automatic-starting.
3. Turn the potentiometer with the help of a little screwdriver very slowly anti-clockwise until it blocks.
4. Let motor run, a little higher than lowest speed, recommended speed 5.000 min^{-1}
5. Turn the potentiometer very slowly and carefully clockwise up to that point the suction unit starts.
6. Setting is finished.